DUOCLEAN, SPRAY PROTECTEUR

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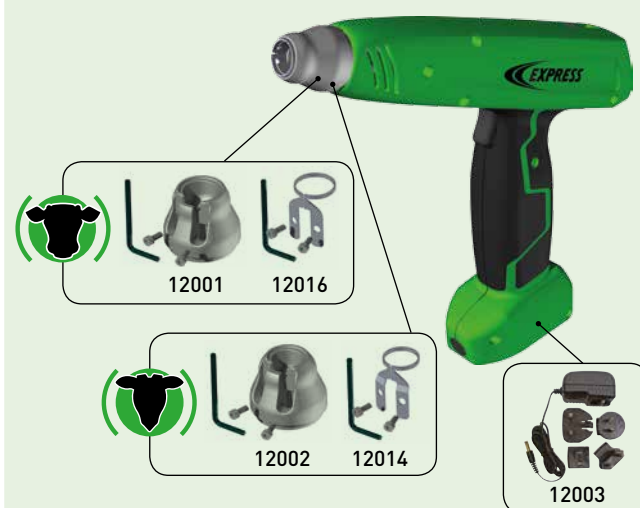
PROTECTIVE SPRAY

- Recommended following dehorning.
- **Relieves** pain
- Produces a **cooling action** on the wound
- **Desinfects**
- Protects: It has a «second skin» effect that allows the wound to breathe
- **Repellent** (especially for flies)
- Water impermeable, does not run and does not wipe off.
- For cattle, goats, sheep, pigs and horses

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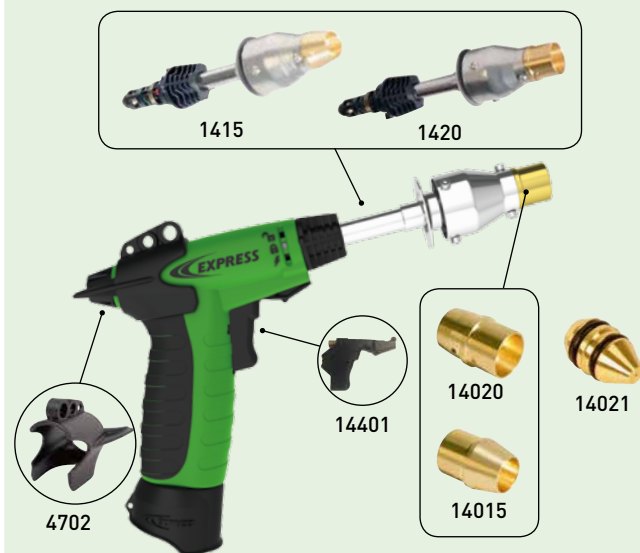
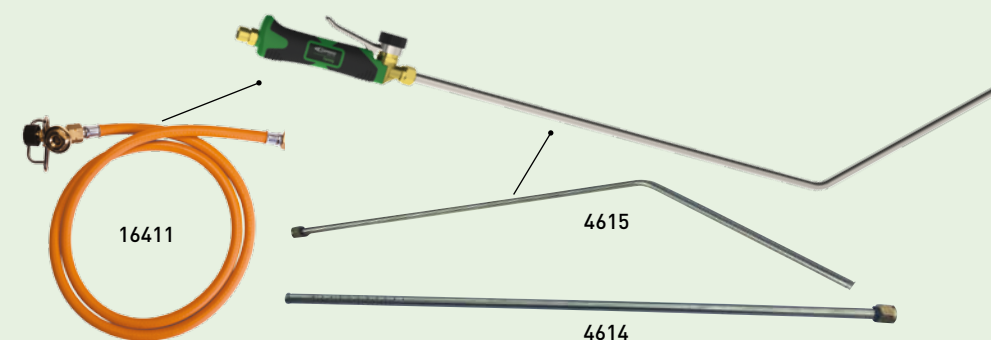
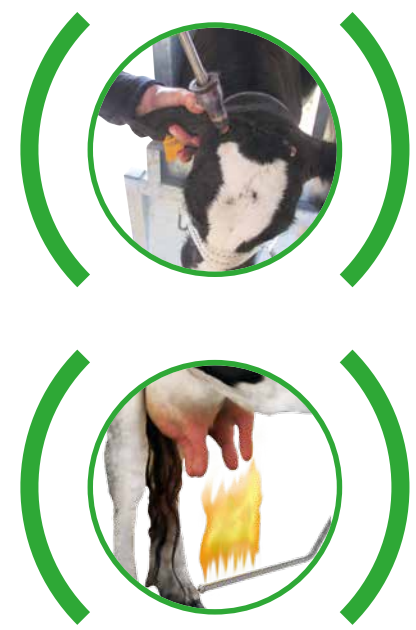
Cleaning udder to improve hygiene before milking avoids **germs, bacteria, mastitis and lost time.**

For more than 15 years, Express has provided the solution: TERRA, the portable thermal hair remover!

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► Keep the flame at a distance of approximately 10 to 20 cm while drawing an "eight" for 2 to 4 seconds.

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EASY • Set up in less than 2 minutes

FAST • Hair removal in less than 4 seconds per cow

SAFE • Easy and safe hair removal due to the **long curved burner**

PAINLESS • Thanks to the «cool flame», the animal feels only a sensation of warmth

HANDY • Cartridge weight 340 g with duration of 2 hours

EFFECTIVE • Reduces the spread of germs and risk of mastitis

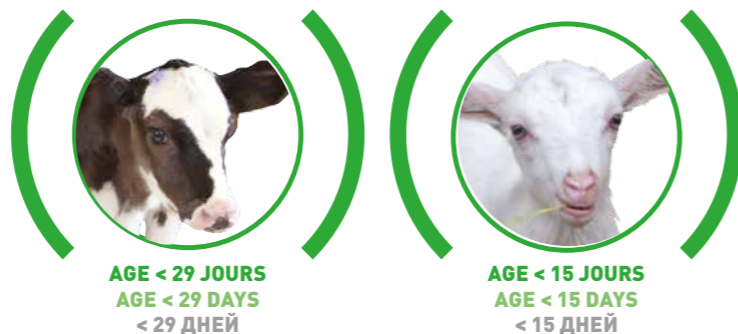
- Faster cleaning of the udder before milking
- Easier teat identification for automated milking
- Improved milk quality

ECONOMICAL • Avoids multiple processing of milk
Compatible with gas cylinder/cartridge

► Handle + angled burner: ± 500 g

► Burner length: 1000 mm
15655: Handle + angled burner + 1.50 m hose + 1 butane/propane cartridge (Cat. No. 556) (Gas cartridge not included).

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• FIABLE

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To avoid risks of injury to animals and farmers, Express provides gas solutions for dehorning young calves and goats:

• FAST

± 5 seconds per horn bud
(no charging time required)

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WHEN TO DEHORN

It is recommended to perform dehorning as soon as possible, before the age of 4 weeks for calves and before 2 weeks for kids.

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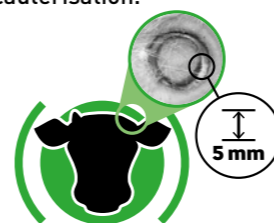
HOW TO DEHORN

BEFORE

- ▶ Immobilize the animal in a cattle crush.
- ▶ Shave around the horn bud before cauterizing in order to:
 - clearly see the location of the future horn bud,
 - avoid getting hair in the wound, which can be a source of infections,
 - clearly see how well the cauterization has been performed.

DURING

- ▶ Apply the tip of the dehorner on the horn bud to cauterize the blood vessels and stop its growth.
- ▶ Do not tear off the horn bud in order to limit the potential infection area. The horn bud will dry and fall off by itself.

AFTER

- ▶ Carefully look at the cauterization ring.
- ▶ A 5 mm-deep groove should be visible around the horn bud. There should be nothing connecting the skin and the horn bud.
- ▶ Apply the antiseptic and repellent spray.

It is recommended to:

Either inject a tranquilizer or, preferably, apply local anaesthetic to the corneal nerve, in order to ensure the comfort of the animal and farmer.

Use a long-acting anti-inflammatory drug to limit harmful consequences of dehorning: refusal to use headlock panels, loss of appetite, isolation.

Avoid dehorning while another treatment (vaccine, etc.) is being administered or a new diet is being given.

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CONTROL YOUR TIME

- ▶ Always ready to use: instantly heats and cools down (Temp. 700 °C / 1300 °F).
- ▶ Automatically stops when the 10-second dehorning cycle is over.

COSMOS BATTERY version

- Cat. No. 124: With Ø 16 mm cattle tip
- Cat. No. 125: With Ø 14 mm goat tip
- ▶ 1 battery charger supplied.
- ▶ ± 760 g

COSMOS MAINS version

- Cat. No. 126: With Ø 16 mm cattle tip
- Cat. No. 127: With Ø 14 mm goat tip
- ▶ Convenient to work with: 6 m cable.
- ▶ ± 560 g

Spare parts

- ▶ Tip formed of a ceramic element and heating wire:
 - 16 mm internal Ø cattle tip
 - 14 mm internal Ø goat tip
- ▶ Charger Cat. No. 12003

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THE BENCHMARK

- ▶ Compact, all-metal
- ▶ No cable/Portable: butane/propane/propene cartridge Cat. No. 446
- ▶ Built-in Piezo ignition
- ▶ Temp.: ± 600 °C / ± 1112 °F
- ▶ Max temp. reached in less than 5 minutes
- ▶ Support stand
- ▶ Tip internal ø 19 mm Cat. No. 11120
- ▶ Additional tips to best adapt to various calf breeds:
 - Tip internal ø 15 mm Cat. No. 11115
 - Tip internal ø 26 mm Cat. No. 11125
- ▶ ± 770 g

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THE PISTOL GRIP

- ▶ Lightweight: ± 470 g
- ▶ No cable/Portable: butane/propane/propene cartridge Cat. No. 446
- ▶ Built-in Piezo ignition
- ▶ Temp.: ± 700 °C / ± 1292 °F
- ▶ Max temp. reached in less than 3 minutes
- ▶ Safety:
 - Lock position
 - Operation position
- ▶ Easy to handle:
 - Non-slip dual-material handle
 - Pistol shape for steady handling
- ▶ Support stand
- ▶ Reversible tip internal ø 17 and 19 mm Cat. No. 14020, to best adapt to various calf breeds
- ▶ Additional tip internal ø 15 mm Cat. No. 14015
- ▶ Self-maintenance: change the trigger/Piezo-ignition and gas nozzle without tools

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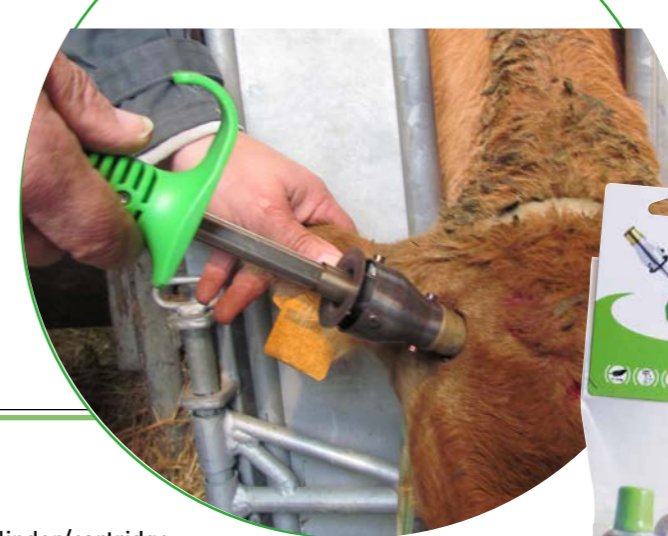
THE CLASSIC

- ▶ Lightweight: ± 300 g
- ▶ Compatible with gas cylinder/cartridge
 - 114/1: Supplied with butane/propane cartridge (Cat. No. 556) and 1.50 m hose
 - 114/2: Supplied with 4.75 m hose and 2-bar regulator
- ▶ Temp.: ± 700 °C / ± 1292 °F
- ▶ Max temp. reached in less than 3 minutes
- ▶ Support stand
- ▶ Reversible tip internal ø 17 and 19 mm Cat. No. 14020, to best adapt to various calf breeds
- ▶ Additional tip internal ø 15 mm Cat. No. 14015

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RESTRAINT CAGE FOR CALVES Cat. No. 47009

- Our cage is designed for dehorning young calves aged 0 to 4 weeks, in order to ensure effective restraint.
- It can be **transported** in a van **without being dismantled**.
- Fully galvanised to ensure a **long lifetime**.
- 1 tray to place the spray, dehorner and clipper.
- **Adjustable restraint ring**: no risk of strangling.
- Front door with 180° opening.
- Large inflatable wheel for easy transport.
- Embossed non-slip floor mat.
- Overall dimensions: 1350 x 1350 x 340 mm.
- Weight: 66.5 kg.

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DEHORNING RESTRAINT SYSTEM SPECIALLY DESIGNED FOR KIDS

Cat. No. 47008

- Can be **adapted** to fit any type of gate.
- Very easy to install.
- **Convenient to work with**.
- Optimal head support.
- **Self-adjustable** according to the kid's head.
- Left-right **reversible**.
- Body fully supported.
- **Ergonomic**.
- Includes a tray to place equipment (clipper, dehorner, spray).
- Weight: 4.5 kg.
- Dimensions of assembled cage: 50 x W 138 x H 29 cm.

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CLIPPER KIT FOR DEHORNING

Réf. 47017

- Includes :
 - 1 clipper
 - 1 brush
 - 1 comb
 - 1 bottle of oil
 - 1 support
- 1 comb size designed to properly clear the area.
- **No cable**.
- Long battery life (100 mins).

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